# AngelSounds JPD- 100S Mini (Smart)

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# AngelSounds JPD- 100S Mini (Smart)

Equipment for monitoring fetal cardiac  
ozone

## Manual

## Product introduction

The AngelSounds JPD-100S Mini (Smart) is a lightweight and portable detector for listening to baby fetus heart sounds.

*Note: The fetus must have at least 16 weeks to function properly. Talk to your doctor about using the device.*

## Contents of the pack

AngelSounds JPD-100S Mini Smart, battery, recording cable, headset, manual.

## Security Warning


Avoid using the device in a potentially explosive, high or very low temperature or humidity environment, near an open fire. We recommend not to use the device outdoors.

Always wipe the gel off the device after monitoring is complete.


Do not use the device if it shows signs of damage!

If you are cleaning the device, always keep it turned off.

## Basics of use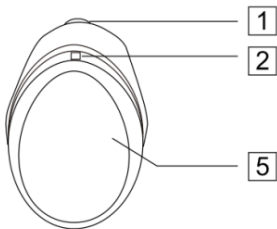

1. Turning On/Off, Adjusting Volume
2. On LED
3. Headphone Connector
4. Signal Transmission
5. Battery cover
6. Audio signal connector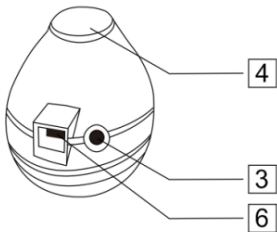

Turn the device on by turning the wheel (1), turning it on is indicated by turning on the diode. Gradually turning the wheel adds volume, turning the other way reduces the sound gradually until the device is completely switched off.

The device offers two connectors. A headphone connector and an audio signal connector. You can use this after


connecting a recording cable to record the sound of fetal heart sounds.

If you don't need to record the audio, just plug in the supplied headphones to listen.

## Battery

Battery is included. It's located right in the device, but it's not connected. Before using it for the first time, the battery must be connected as shown in the following image.



Remove the cover (5) and connect the battery to:

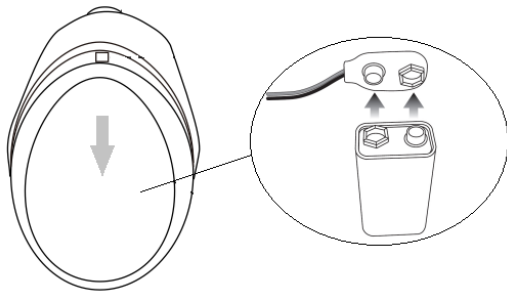## The procedure for finding the foetal heart

Apply some gel to the signal transfer point (4) and place it on your stomach. Move it slowly and try to find the heart of the fetus. After the sound is found, continue to move the probe slowly and look for the place where the sound signal will be clearest.

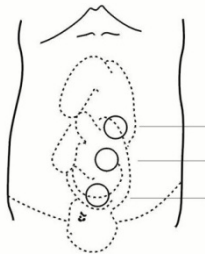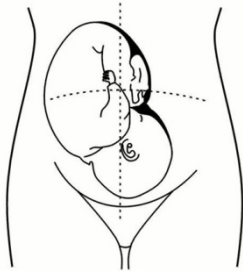You can see the usual fetal

position in the following picture. However, the position of the foetus may vary depending on the degree of pregnancy and other conditions.

Count the number of ornaments per minute to get information about heart rhythm. We recommend that you repeat the counting multiple times.

## Audio Recording

You can also record these sounds while listening to the heart sounds. All you have to do is plug the recording cable into the mini-USB connector, plug the other end into the device on which the sound will be recorded and start recording.

You can also run AngelSounds (only for the Smart version), once you connect the other end of the cable to your phone, a window will automatically open to record the sound of fetal heart sounds.

## Disposal

The product contains batteries and/or recyclable electrical waste. In order to protect the environment, do not dispose of  the product with conventional waste, but hand it over for recycling at an electrical waste collection point.

## Declaration of Conformity

By this importer, Satomar s.r.o (Karlova 37, 614 00 Brno) declares that AngelSounds JPD 100S Mini and AngelSounds JPD 100S Mini Smart complies with all the essential requirements of the EU directives that apply to it.

The complete text of the Statement of Conformity can be downloaded at <http://www.satomar.cz>. Copyright © 2020 Satomar, s. r. o. All rights reserved.

Appearances and specifications can be changed without notice. Printing errors reserved.

The most up-to-date version of the manual can be found at [www.mobilnipodpora.cz](http://www.mobilnipodpora.cz).

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Copyright © 2020 Satomar, s. r. o. Alle Rechte vorbehalten.