AngelSounds

JPD-200S

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AngelSounds

JPD-200S

Fetal heart rate monitoring equipment

Manual

Product presentation

AngelSounds JPD-200S is a light and portable detector for listening to the heart sounds of the baby's fetus.

Note The fetus must be at least 20 weeks old to function properly. Consult a doctor before using the device.

Package contents

AngelSounds JPD-200S, jack-jack cable, headphones, batteries, manual

Safety notice




Avoid using the device in an environment with a potential risk of explosion, in an environment with high or very low temperature or humidity, near an open flame. We recommend that you do not use the device outdoors.

Always wipe the gel off the device after monitoring.

Do not use the device if it shows signs of damage!

Always keep the device switched off when cleaning.

Basics of use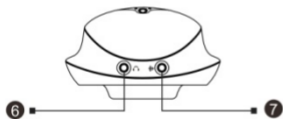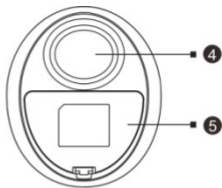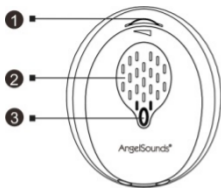


- 1 - On / off; volume adjustment
- 2 - Loud speaker
- 3 - Status indicator
- 4 - Signal transmission
- 5 - Battery cover
- 6 - Headphone connection
- 7 - Connection for recording cable

Turn on the device by turning the wheel (1), switching on is indicated by the LED lighting up. Gradually turning the wheel increases the volume, turning to the opposite side gradually reduces the sound until the device is completely turned off.



You can hear heart sounds either from the loudspeaker or by connecting headphones to the appropriate jack (6).

Audio can also be recorded. Connect the cable with the device and the mobile phone and start recording audio.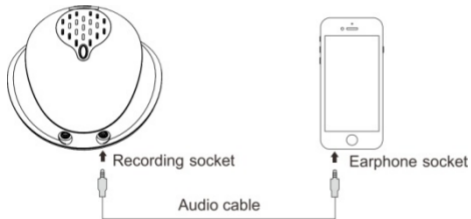

Battery

The battery is included in the package. It is located directly on the device, but is not connected. Before first use, the battery must be connected as shown below. Remove the cover (5) and connect the battery: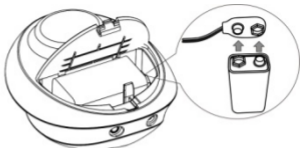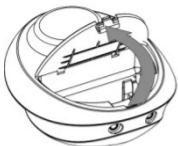


Indicator states

Indicator status	Operating status
Solid green	The device is on
Flashing green	The device is switched on and picks up the sound of the fetus
Flashing orange	The device needs recharging

The procedure for finding the heart of the fetus

Apply a little gel to the probe (4) and place it on the abdomen. Move slowly with it and try to find the heart of the fetus. After finding the sound, continue to move the probe slowly and look for the place where the sound signal will be clearest.

You can see the usual position of the fetus in the following picture. However, the position of the fetus may vary depending on the stage of pregnancy and other conditions.

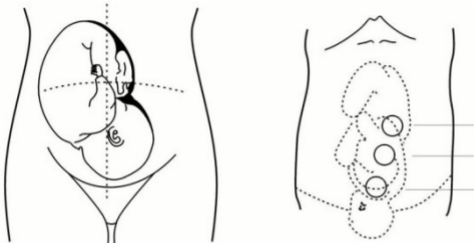Technical Specifications

Dimensions: 100 mm × 85 mm × 50 mm; 0.15 kg

Acoustic frequency: 3.0MHz ± 5%

Effective recording area: 7.4 cm²

Good operation: 4-6 hours

Battery: 9V

Water resistance: IP22

Conditions of use: 5-40 °C

Disposal

The product contains batteries and/or recyclable electrical waste. In order to protect the environment, do not dispose of the product with conventional waste, but hand it over for recycling at an electrical waste collection point.

Declaration of Conformity

By this importer, Satomar s.r.o (Karlova 37, 614 00 Brno) declares that product complies with all the essential requirements of the EU directives that apply to it.

The complete text of the Statement of Conformity can be downloaded at <http://www.satomar.cz>. Copyright © 2020 Satomar, s. r. o. All rights reserved.

Appearances and specifications can be changed without notice. Printing errors reserved.

The most up-to-date version of the manual can be found at www.mobilnipodpora.cz.

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