Jumper JPD-500F - pulzní oxymetr

Manuál


Obsah balení

Oxymetr, pouzdro, 2× AAA baterie, šňůrka
s poutkem, manuál

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Battery cover
– kryt baterie
Battery cover
lock – zámek
krytu

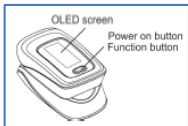OLED screen
 – displej
 Power on
 button /
 Function
 button –

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Popis informací na displeji

Vlevo nahoře – symbol saturace krve

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
Manual

Contents of the package

Oximeter, case, 2× AAA battery, drawstring with loop, manual

Description of the device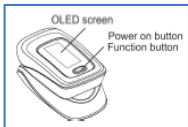

Description of the information on the display

Top Left - Blood Saturation Symbol

Centre Left - Saturation Measured

Top Middle - Battery Status

Middle - PI Symbol

Bottom Middle - Measured PI Value

Top Right – Pulse Measurement Symbol

Right – Measured Pulse

Bottom Of Display - Graph

Note the battery symbol and bluetooth symbol may not be displayed on the display at all times.

Set up your device

Briefly hold the button to turn on the device.

Press the button again briefly to change the display orientation.

If you hold the button longer, you will be taken to the settings. Here you move at the short touch of a button, changing the options by holding the button for a longer time.

In settings, you can set an alarm when abnormal values are detected, adjust the sound, display brightness, and restore to factory settings

Use of the device

Insert the index finger into the device, as shown in the figure. Turn on the device. After a few seconds, the measured value appears on the display.

Note: When measuring, try to be calm and do not move your hand over whose finger measurements are made. Otherwise, the measurement may be inaccurate.

Link to an app

Open JumperHealth and run the oximeter menu in it. Turn on the device, it will automatically connect to your mobile phone. The measured values are displayed on the mobile display in real time. In the History tab, you can see the measurement history.

Replacing batteries

Release the lock of the back cover and then slide the cover out. Insert two AAA batteries.

Safety instructions

Do not subdue the device in water.
Clean the device with a gently damp cloth.
Do not use the device if it shows signs of any damage.
If you plan not to use the device for a long time, remove the batteries.

Do not leave the device in direct sunlight.

Disposal

The product contains batteries and/or recyclable electrical waste. In order to protect the environment, do not dispose of the product with ordinary waste, but hand it over for recycling to a collection

point of electrical waste.

Declaration of conformity

The importer, Satomar s.r.o (Karlova 37, 614 00 Brno), declares that the product meets all the essential requirements of eu directives that apply to it.

The full text of the Declaration of Conformity can be downloaded <http://www.satomar.cz>. Copyright © 2021 Satomar, s. r. o. All rights

reserved.

Appearance and specifications may be changed without notice. Print errors reserved.

The most up-to-date version of the manual can always be found www.mobilnipodpora.cz.

Jumper JPD-500F – pulse oximeter


Manual

Contents of the package

Oximeter, case, 2× AAA battery, drawstring with loop, manual

Description of the device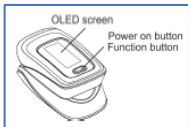

Description of the information on the display

Top Left - Blood Saturation Symbol

Centre Left - Saturation Measured

Top Middle - Battery Status

Middle - PI Symbol

Bottom Middle - Measured PI Value

Top Right – Pulse Measurement Symbol

Right – Measured Pulse

Bottom Of Display - Graph

Note the battery symbol and bluetooth symbol may not be displayed on the display at all times.

Set up your device

Briefly hold the button to turn on the device.

Press the button again briefly to change the display orientation.

If you hold the button longer, you will be taken to the settings. Here you move at the short touch of a button, changing the options by holding the button for a longer time.

In settings, you can set an alarm when abnormal values are detected, adjust the sound, display brightness, and restore to factory settings

Use of the device

Insert the index finger into the device, as shown in the figure. Turn on the device. After a few seconds, the measured value appears on the display.

Note: When measuring, try to be calm and do not move your hand over whose finger measurements are made. Otherwise, the measurement may be inaccurate.

Link to an app

Open JumperHealth and run the oximeter menu in it. Turn on the device, it will automatically connect to your mobile phone. The measured values are displayed on the mobile display in real time. In the History tab, you can see the measurement history.

Replacing batteries

Release the lock of the back cover and then slide the cover out. Insert two AAA batteries.

Safety instructions

Do not subdue the device in water.
Clean the device with a gently damp cloth.
Do not use the device if it shows signs of any damage.
If you plan not to use the device for a long time, remove the batteries.

Do not leave the device in direct sunlight.

Disposal

The product contains batteries and/or recyclable electrical waste. In order to protect the environment, do not dispose of the product with ordinary waste, but hand it over for recycling to a collection

point of electrical waste.

Declaration of conformity

The importer, Satomar s.r.o (Karlova 37, 614 00 Brno), declares that the product meets all the essential requirements of eu directives that apply to it.

The full text of the Declaration of Conformity can be downloaded <http://www.satomar.cz>. Copyright © 2021 Satomar, s. r. o. All rights

reserved.

Appearance and specifications may be changed without notice. Print errors reserved.

The most up-to-date version of the manual can always be found www.mobilnipodpora.cz.

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heruntergeladen werden. Copyright © 2021 Satomar, s. r. o. Alle Rechte vorbehalten.