Jumper Medical JPD-HA121

Manuál

Obsah balení

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Manual

Contents of the package

Pressure gauge, sleeve, connecting hose, travel packaging, manual, 4× AA batteries.

Description of the device

The sleeve tube connects to the connector on the left side of the device.

Below the display there are three control buttons – switching between users, functional button, memory access button.

Charging

By default, the device works with four AA batteries. Alternatively, a microUSB connector located on the right side can be used.

Switch between users

You don't need to turn on the device to switch between user A and B. Just briefly press the right control button.

Set a date and time

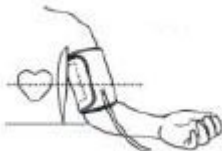
In the off state, hold down the user switch button for a long time. You then briefly switch between values, pressing the right button to set them.

Measurement instructions

We recommend skipping more food, caffeine or exercise 30 minutes before measuring.

Check that the sleeve-to-sleeve hose is reliably connected to the device.

The sleeve of the pressure gauge must be in direct contact with the skin. Can't be measured through clothes.

Securely
fasten the
sleeve of
the
pressure
gauge

around the arm (ideally use your left arm), above the elbow. Rest your hand on a solid surface, see figure.

The measurement begins when the power button on the pressure gauge is pressed.

At the end of the measurement, the sleeve is automatically blown out and the measured values (systolic and diastolic pressure and pulse) are displayed, see figure.

History of records

The device can store 99 records for two different people. The next time you measure, the oldest records are automatically overwritten. To erase memory, right-click

to open the measurement history and hold the right button for a long time.

Connect to a mobile app

Open JumperHealth and the pressure measurement section. Start pressure measurement on the pressure gauge. The device connects with each other. The link works on the basis of Bluetooth. In the History tab, you can see the measurement history.

Disposal

The product contains batteries and/or recyclable electrical waste. In order to protect the environment, do not dispose of the product with ordinary waste, but hand it over for recycling to a collection point of electrical waste.

Declaration of conformity

The importer, Satomar s.r.o (Karlova 37, 614 00 Brno), declares that the product meets all the essential requirements of eu directives that apply to it.

 The full text of the Declaration of Conformity can be downloaded <http://www.satomar.cz>.

Copyright © 2021 Satomar, s. r. o. All rights reserved.

Appearance and specifications may be changed without notice. Print errors reserved.

The most up-to-date version of the manual can always be found www.mobilnipodpora.cz.

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