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Viatom Wellue –
portable Mash inhaler
Manuál

Product description

Portable mesh inhaler with two inhalation modes, adjustable intensity, removable medicine container, automatic cleaning technology, non-stop operation for up to 60 minutes and low weight and noise..

Safety instructions

Consult your doctor about using the device.

Consult your doctor about using a particular inhalation solution.

If the devices are used by children, they should be supervised.

Do not use the device if it has been damaged or has come into contact with water.

Keep equipment clean.

Do not pour tap water into the device.

Product description

1 - spray unit

2 – Cap

3 - Medicine container

4 – On


5 - main part of the device

6 - USB charging

7 – Mask


8 – mouthpiece

10 - Adjustment of intensity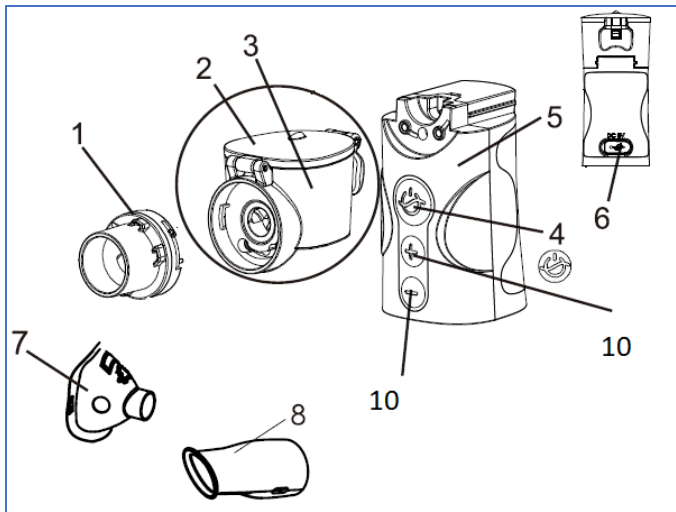


Mounting

The equipment is delivered disassembled. Follow the pictures when assembling.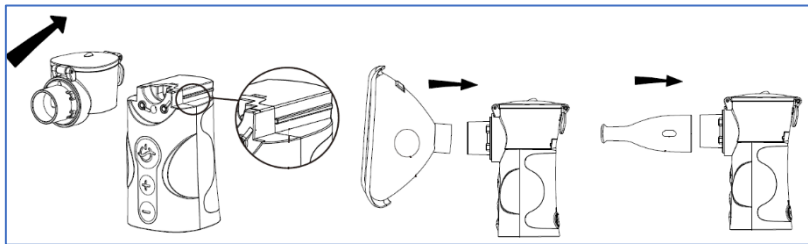

Before switching on the device, add the inhalation solution to it as directed by your doctor. When opening the lid afterwards, be careful not to spill the solution.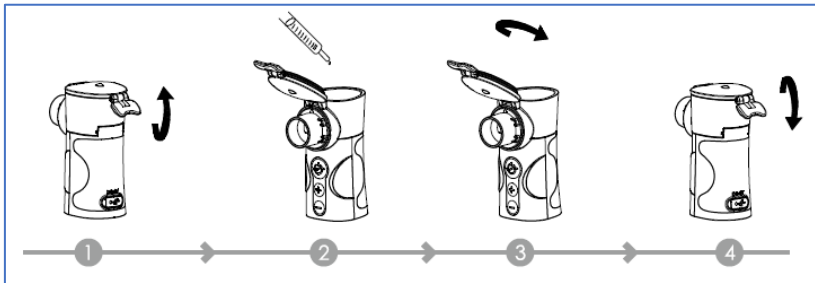

Charging

The device is charged using a USB cable and charger (not included). Charger specifications should be DC 5.0V 1.0A.

Low battery status is indicated by a flashing flash diode (approx. it stops after 5 minutes of flashing).

The charging time is approximately two hours. The operating time on the battery is approximately 60 minutes.

While charging, the flash diode flashes when the device is fully charged, the flashing stops.

Instructions for use

Green light on - manual mode

Green light changes intensity - ultrasonic mode

Green light flashes three times - mode switching indication (must hold on button for 3 seconds)

Blue light flashes five times - low battery capacity

The orange light flashes ten times - inhalation solution is missing and device switched off


Blue light blinks – charging

Blue light on - fully charged

Green light flashes 10 times - end of 10-minute mode

The device offers two inhalation modes, press the power button for three seconds to switch them on. Switching is indicated by flashing green light. In manual mode, the green light shines continuously, using the + and - buttons to change the intensity of the inhalation. In ultrasonic mode, the green light amplifies and dims its intensity.


You can inhale in three ways as shown in the picture. Straight, through the mask and through the mouthpiece.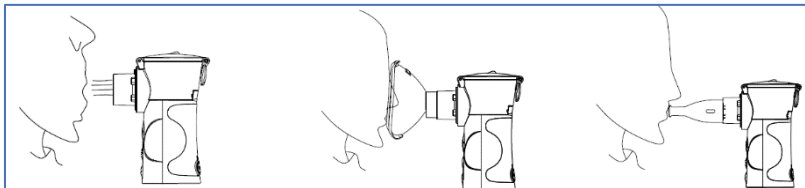

The device is set to allow a maximum inhalation time of 10 minutes, then switched off. If you need to continue inhaling, turn the device back on.

To do manual shutdown, press the power button briefly.

Discard the remainder of the inhaled solution after inhalation, disassemble the pieces of the device (see picture) and clean them.

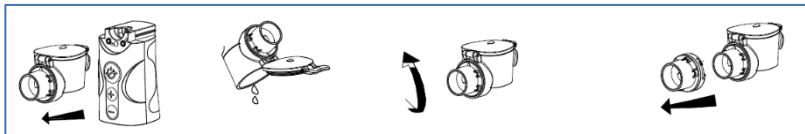Cleaning

After each use, the apparatus and its component parts must be cleaned and disinfected. Do not clean while the device is on or charging. Only reassemble the device if each component is completely dry.

Never submerge or soak the main unit in water. Use a damp cloth, etc., for cleaning.

Disposal

The product contains batteries and/or recyclable electrical waste. In order to protect the environment, do not dispose of the product with conventional waste, but hand it over for recycling at an electrical waste collection point.

Declaration of Conformity

By doing so, the importer, Satomar, s.r.o., declares that the equipment complies with all the essential requirements of the EU directives that apply to it.

The complete text of the Statement of Conformity can be downloaded at <http://www.satomar.cz>.

Copyright © 2020 Satomar, s. r. o. All rights reserved.

Appearances and specifications can be changed without notice. Printing errors reserved. The most up-to-date version of the manual can be found at www.mobilnipodpora.cz.

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Copyright © 2020 Satomar, s. r. o. Všechna práva vyhrazena.
Copyright © 2020 Satomar, s. r. Ö. Alle Rechte vorbehalten.
Copyright © 2020 Satomar, s. r. o. Minden jog fenntartva.